
Quickstart Guide

Main Unit

CT40

SUNTECH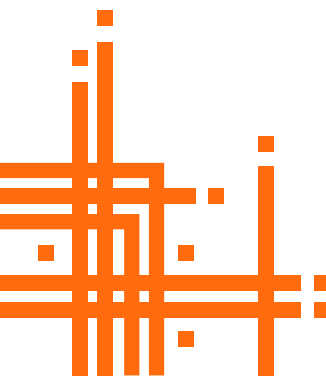

SunTech Medical, Inc.
5827 S Miami Blvd, Suite 100
Morrisville, NC 27560-8394, USA
Tel: + 1.919.654.2300
1.800.421.8626
Fax: + 1.919.654.2301
SunTechMed.com

EMERGO Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

International Associates Limited
Centrum House, 38 Queen Street, Glasgow,
Lanarkshire, G1 3DX, UK
UKRP@ia-uk.com

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

SunTech Medical, Ltd.
105 HuanGuan South Road, Suite 15 2-3/F
DaHe Community Guanlan,
LongHua District, Shenzhen
GuangDong PRC 518110
Tel: + 86-755-29588810
+ 86-755-29588986 (Sales)
+ 86-755-29588665 (Service)
Fax: + 86-755-29588829
SunTechMed.com.CN





Download operating instructions for this product by visiting suntechmed.com/library.
View the instructions for this product using Adobe Acrobat Reader, or any compatible PDF reader, on any computer or mobile device with an internet connection. To request a paper copy of current or historical operating instructions at no additional cost and within 7 calendar days of receipt of the request, please contact SunTech Customer Service at CustomerSupport@SunTechMed.com or by calling 800.421.8626.

82-0143-00-MA RevF

SUNTECH

1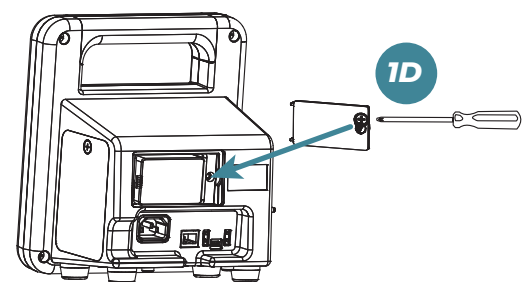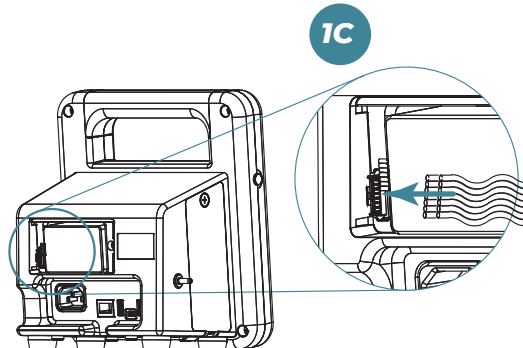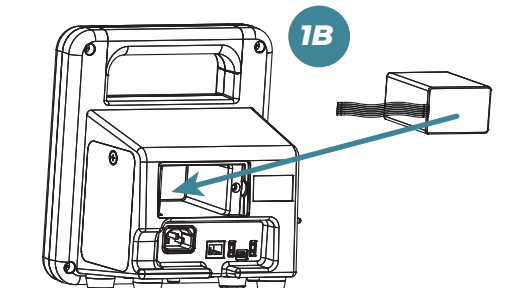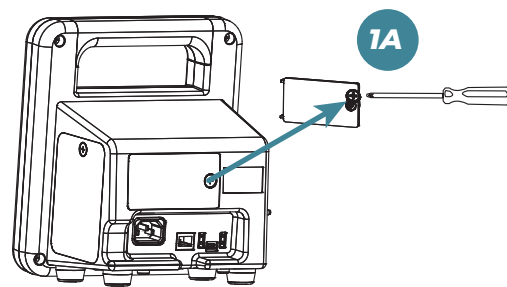

Battery

- 1A. Loosen screw and remove battery door
- 1B. Insert battery with cable facing out
- 1C. Plug end of battery cable into connector
- 1D. Replace battery door and tighten screw

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
4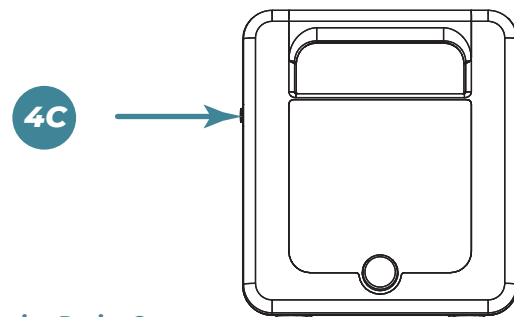

Turning Device On

- 4A. Insert and connect battery (see step 1)
- 4B. Connect AC power cord (see step 2A)
- 4C. Press power button (do not hold down for more than 5 seconds)
- 4D. Set the time and date

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5

To Access Advanced Configuration Settings

- 5A. Please visit <http://www.suntechmed.com/ct40-drivers> to download and install the driver required for connecting the CT40 to a PC. The driver MUST be installed in order to access the application
- 5B. Connect Device to PC, laptop, or tablet using Mini USB (see step 2E)
- 5C. Open browser, enter device address (<http://model260.local>)
- 5D. Enter default password when prompted. Default passwords are: "service" for full access to configure the device "medical" for access to patient measurements
- 5E. When prompted, enter a new password.

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Download operating instructions for this product by visiting [suntechmed.com/library](http://www.suntechmed.com/library). View the instructions for this product using Adobe Acrobat Reader, or any compatible PDF reader, on any computer or mobile device with an internet connection. To request a paper copy of the IFU at no additional cost, please contact SunTech Medical, Inc. at 919-654-2300. Printed copies will be attempted to be delivered within 7 calendar days of requests.

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