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**SunTech Medical**  
The Difference in Clinical Grade™

0413  
82-0546-00 MA Rev. A

For assistance, contact:  
Pour une assistance, contactez:  
Beratung und Unterstützung erhalten Sie hier:  
Per assistenza contattare:  
Para obtener asistencia, póngase en contacto con:  
Para assistência entre em contato com:  
如需支持，请联系：

SunTech Medical, Inc.  
507 Airport Boulevard, Suite 117  
Morrisville, NC 27560 USA  
Tel: 800.421.8626  
919.654.2300  
Fax: 919.654.2301  
Email: CustomerSupport@SunTechMed.com  
Web: www.SunTechMed.com

To register this device, go to  
[www.SunTechMed.com/register](http://www.SunTechMed.com/register)

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Download operating instructions for this product by visiting [suntechmed.com/ct50-documents](http://suntechmed.com/ct50-documents). View the instructions for this product using Adobe Acrobat Reader, or any compatible PDF reader, on any computer or mobile device with an internet connection.

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### Welcome to the SunTech CT50!

The CT50 vital signs monitor easily provides reliable blood pressure (BP), SpO<sub>2</sub>, and temperature measurements for neonatal, pediatric, and adult patients.

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## Introduction au CT50

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## Guia Rápido do CT50

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- English**
- Physiological alarm visual indicator LED's.
    - High level alarm: the lamp quickly flashes red.
    - Medium level alarm: the lamp slowly flashes yellow.
    - Low level alarm: the lamp lights yellow without flashing.
  - LCD Touch screen
  - SpO<sub>2</sub> connector
  - NIBP connector
  - USB connector x 2
  - Power button
  - Battery charging indicator LED
  - Battery indicator LED. Status of the LED is specified as follows:
    - Green: When the AC mains connected.
    - Orange: When the AC mains not connected and monitor is powered by battery.
    - Off: When the AC mains not connected.
  - Well for 20-count Probe Cover box
  - Covidien Filac 3000 temperature probe
  - Print Recorder
  - Grounding terminal
  - Nurse call connector
  - AC power connector (input)
  - DC power connector (output)
  - USB socket x 2
  - Ethernet LAN Network connector

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- English**
- Clinician Information  
Note: In monitoring mode, you must login a clinician prior to selecting the patient.
  - System Time, Date, and Network status
  - Battery Status
  - Device Alarm Message Bar
  - Measurement Display Area
  - Patient Information
  - Menu Tabs: navigate the device menu.
    - MEASURE: Display vital sign parameters.
    - PATIENT
    - REVIEW: Historical patient data.
    - ALARM: Adjust alarm parameters
    - SETTINGS
  - Shortcut Icons
    - Help key
    - Alarm pause key
    - Shortcut key to reset the alarm
    - Shortcut key to print
    - Shortcut key to start/stop NIBP measurement
    - Shortcut key to standby mode
    - Shortcut key to the home screen
  - Save Icon: Press to save the current measurement data for the current patient.

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