

AccuWinPro forwards compatibility upgrade installation

These instructions will update certain AccuWinPro installations to talk to both older Oscar 2 models and newer Oscar 2 models.


These installation instructions apply to the following AccuWinPro software versions:

- AccuWinPro v3.x
- AccuWinPro SE


Instructions

Note: Ensure the AccuWinPro application is closed before following these instructions and any ABPMs are removed from the PC.


1. Install the driver located in the Driver Install Director accompanying these instructions
 - a. Open the driver install folder
 - b. Choose either the x86 or x64 driver based on your PC configuration
 - c. Double-click on the dpinst.exe file to start the install
2. Locate the AccuWinPro install directory. This is usually in C:\AWPv3 or C:\AWPSE

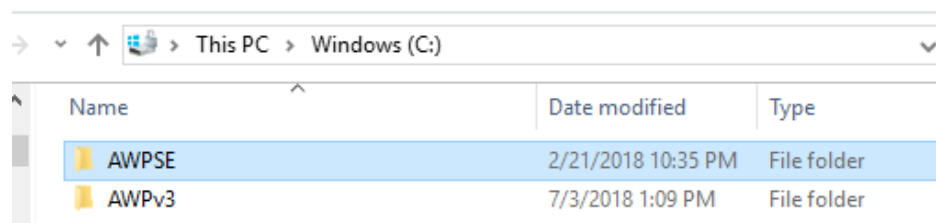

3. Locate the o2dllprj.dll file in the install folder

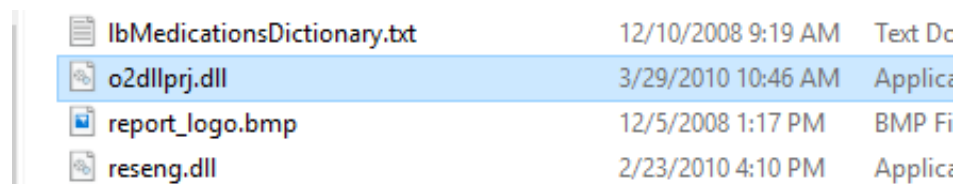

4. Rename the file to o2dllprj-old.dll

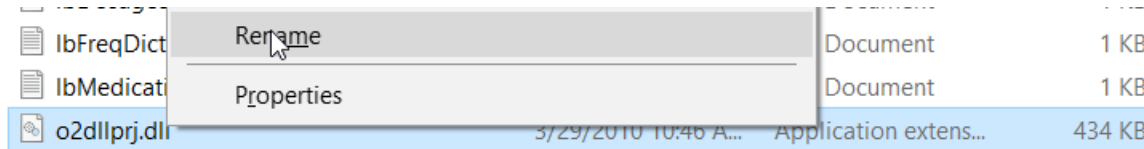

- a. Right-click on the file
- b. Select 'rename' in the context menu
- c. Add -old to the file name before the file extension (**be careful to not delete the file name completely!**)
- d. Press 'enter' to complete the file name change


lbMedicationsDictionary.txt	12/10/2008 9:19 AM	Text Doc
o2dllprj-old.dll	3/29/2010 10:46 AM	Applicat
report_logo.bmp	12/5/2008 1:17 PM	BMP Fil
reseng.dll	2/23/2010 4:10 PM	Applicat

5. Copy the o2dllprj.dll file included with these instructions to the AccuWin install directory.

lbMedicationsDictionary.txt	12/10/2008 9:19 A...
o2dllprj.dll	3/29/2010 10:46 A...
o2dllprj-old.dll	3/29/2010 10:46 A...
report_logo.bmp	12/5/2008 1:17 PM


6. Launch AccuWinPro and ensure device communication is functional by programming a device to start a study. If no error messages occur, the change has been completed successfully.

7. If issues in communications develop:
 - a. Close AccuWinPro
 - b. Delete the new o2dllprj.dll DLL file

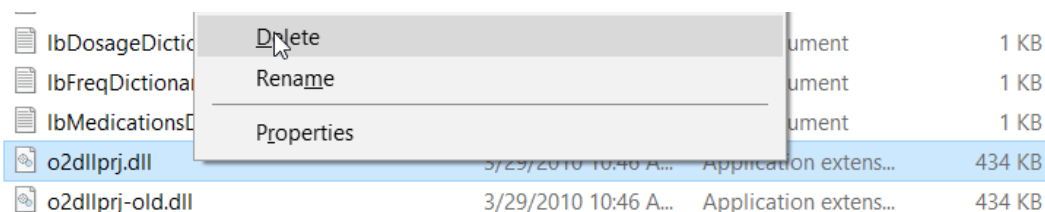

- c. Rename the previous o2dllprj-old.dll to o2dllprj.dll

507 Airport Blvd., Suite 117, Morrisville, NC 27560-8200 USA | 919-654-2300 | Fax 919-654-2301 | SunTechMed.com

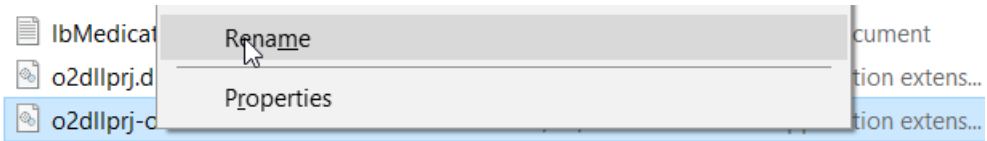

d. Relaunch AccuWinPro