


Biosound Esaote Formula for Archimed Tango Interface Notes

You can setup the Tango with your stress system using the following directions. You will need the ECG trigger and RS-232 cables specified below.

- The ECG trigger cable is required for the Tango monitor to be able to take a BP measurement.
- You will need the RS-232 cable to establish communication between the Tango and your stress system. This cable allows your stress system to prompt the Tango when it needs a BP measurement. In addition, it allows the BP measurements taken by the Tango to be transferred to your stress system display and reports.

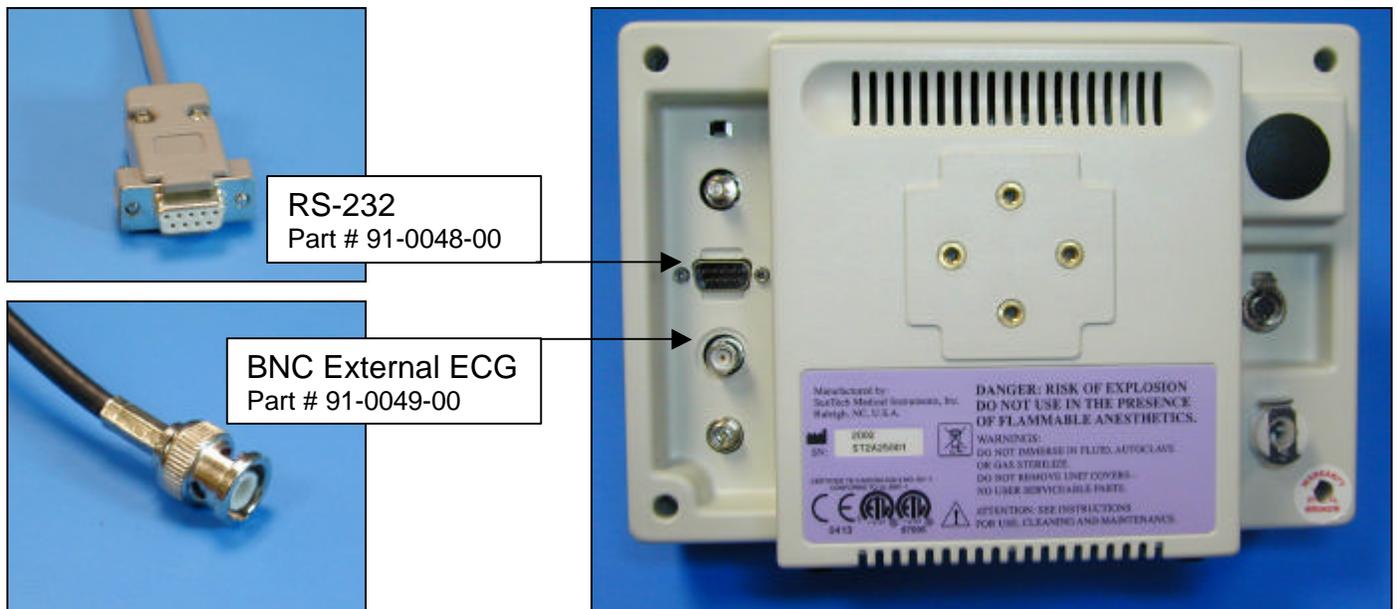

1. Plugging in the ECG trigger cable
Connect: the ECG trigger cable, SunTech part #91-0049-00 (BNC to 9 pin male)
From: the BNC External ECG connection on the rear panel of the Tango (BNC)
To: the AUX port on the back of the stress system (9 pin male)
2. Plugging in the RS-232 cable
Connect: the RS-232 cable, SunTech part #91-0048-00 (9 pin male to 9 pin female)
From: the RS-232 connection on the rear panel of the Tango (9 pin female)
To: the port on the back of the stress system (9 pin male)
3. Setting up the Tango monitor
 - a. Turn on the Tango using the switch at the rear top left corner of the Tango.
 - b. Once the operating screen is displayed, press the *MENU/SELECT* button. Use the UP or DOWN buttons to select **Utilities** and press the menu button.

- c. Use the arrow buttons to select **Device** and press the MENU/SELECT button to edit. Choose **BIOSOUND** using the arrow buttons and press the select button to confirm the choice.
 - d. Select **ECG trigger** and press the select button to edit. Choose **DIGITAL**↑ and press the select button to confirm the choice.
 - e. Select **Exit** twice to return to the operating screen.
4. Setting up the stress system (Note: requires software version 2.1 or greater)
- a. From the main monitor screen, press **MENU** located on the far right of the control panel. This will display the menu.
 - b. Select **Exam Preset** and confirm by pressing “ Do” on the keyboard.
 - c. Highlight **Protocol** and press “Do” and then using the roller ball, select the appropriate protocol that you want to add the BP option (e.g. Bruce) to and accept using “Do”.
 - d. Using the rollerball, highlight **Auto** and press “Do”.
 - e. Set the **BP Device** to **SunTech 4240** and the **Port** as **Com 1**.
 - f. In **Protocol**, set when you want the reading to be added to the stage report, e.g. 2 minutes and the set the reminder to 45 - 50 seconds. This will trigger the Tango the reminder time before the reading is expected by the host.
 - g. Confirm all settings by pressing "Do" prior to exiting from the screen by pressing the "Undo" key.